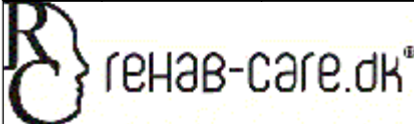



ITEM NO.	PART NUMBER	Description	QTY.
1	L53-H	Glideklods	4
2	B2-700	Bobath topamme 700	1
3	0404001	Løftemotor MOT-38	1
4	M4	Saks udvendig	1
5	L-3000	Bundramme saks	1
6	M5	Saks indv.	1
7	L45	Plast bøsning	4
8	L6-A	Løfte laske	2
9	L16-2	Afstandsør ø16x13	4
10	L-3000-03	Forbindelsesplade saks	2
11	SKA28-1	Medbringer bremse	2
12	ISO 7380 - M12 x 30 - -- 30N		4
13	ISO 7380 - M8 x 20 -- - 20N		4
14	Hexagon Nut ISO - 4032 - M8 - W - N		1
15	L-3010-01	Bremse stang	1
16	Pedal-højre	Bremsepedal	1
17	L-3000-50	Hjul holder	4
18	Pedal-venstre	Bremsepedal	1
19	60X30 Flad oval	Plastprop grå 60x30 f-oval	4
20	5444-46PJP125R36- 34	Hjul ø125	4
21	L500	Løftetravers saks	1
22	L42-20	Bremsestang til pedal	1
23	L42-10	Bremsestang L=580	1

Rev	Date	Note
A	09-06-16	ny bundramme

 Cypresvej 3 -7400-Herring Denmark WWW.rehab-care.dk	Scale: 1:10
	Paper: A3
	Sheet 1 of 2

Description: Bobath leje 200 x 70

Surface treatment:	Color:	
Drawn By: MG	Drawn Date: 27-02-19	
Weight: 37130.64	State: - ok	Material:

 Tolerances ISO 2768-mK	Drawing No.: B100-70	Rev. A
---	----------------------	--------

