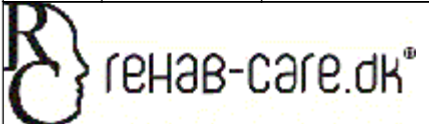



ITEM NO.	PART NUMBER	Description	QTY.
1	Pedal-venstre	Bremsepedal	1
2	Pedal-højre	Bremsepedal	1
3	2474DYK125R26-28S45	Hjul 125mm	4
4	L53-H	Glideklods	4
5	B3-hynde	Hynde for bobathleje	1
6	B2-700	Bobath topramme 700	1
7	L1-B	Hjulramme	1
8	L5-B	Løfte travers	1
9	L4A	Saks udv.	1
10	L6-A	Løfte laske	2
11	CB09	Linak styreboks	1
12	LA31 bund 250-MOT-15	Linak motor 250mm	1
13	L41	Bremsestang L=663	1
14	SKA28-1	Medbringer bremse	2
15	L42	Bremsestang L=620	1
16	L13A	Bremsestang	1
17	SN-08x20	Jern nitte ø8x20	2
18	STAR-LOCK-11mm Capped	Star lock	2
19	M10x60 ISB	Rundhoved M10x60	1
20	28x20x1.0 passkive		2
21	M5	Saks indv.	1
22	ISO 7380 - M12 x 20 - -- 20N		10
23	L16-2	Afstandsør ø16x13	10
24	L45	Plast bøsning	10
25	ISO 7040-M10-N		1
26	30X20 PROP	Plast prop	2
27	L13-2	Bremse stang	1

Rev	Date	Note

 Cypresvej 3 -7400-Herning Denmark WWW.rehab-care.dk	Scale: 1:10
	Paper: A3
	Sheet 1 of 2
Description: Bobath leje 200 x 70	
Surface treatment:	Color:
Drawn By: MG	Drawn Date: 17-12-15
Weight: 136167.28	State: - ok
Material:	
 Tolerances ISO 2768-mK	Drawing No.: B100-70
	Rev.