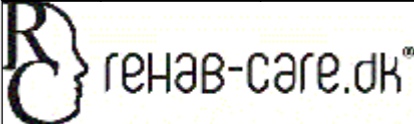



ITEM NO.	PART NUMBER	Description	QTY.
1	BL-H-1000	Benløfter underdel	1
2	BL-H-2000	Arm for benløfter	1
3	BL-2-1008	Glideskinne POM	4
4	80X80 PROP	Plast prop 80x80 hvid	1
5	Hexagon socket countersunk head screw ISO 10642 - M5 x 10		8
6	Hexagon socket countersunk head screw ISO 10642 - M6 x 8		2
7	BL-2-1018	Top bolt for benløfter	1
8	BL-2-1020	Glide skinne yderrør POM	4
9	Hexagon socket button head screw ISO 7380 - M5 x 10		8
10	Hexagon socket button head screw ISO 7380 - M8 x 10		1
11	BL-2-1021	Afstandsør benløfter	2
12	la28-750	MOT-74	1
13	BE-7001	Støtteklods for LA28	1
14	BAT-12		1
15	Hexagon socket head cap screw ISO 4762 - M10 x 50		1
16	Pan head screw ISO 7045 - M5 x 12		4
17	S70-10x15	Gevind tap for hjul	4
18	5325 PJP 075 L51-10	Ø75 hjul m bremse	4
19	Nut DIN 1587 - M10		5

Rev	Date	Note

 Cypresvej 3 -7400-Herring Denmark WWW.rehab-care.dk	Scale: 1:20
	Paper: A3
Sheet 1 of 1	
Description: Ben løfter H-model 750mm	
Surface treatment:	Color: RAL9002
Drawn By: MG	Drawn Date: 23-10-12
Weight: 30158.45	State: - ok
Material:	
 Tolerances ISO 2768-mK	Drawing No.: BL-H
	Rev.