
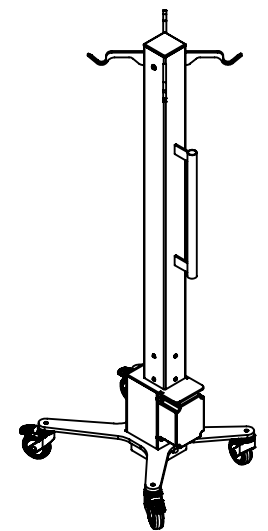



ITEM NO.	PART NUMBER	Description	QTY.
1	BL-H5-100	Dropstativ underdel	1
2	BL-H5-200	Arm for dropstativ	1
3	BL-2-1008	Glideskinne POM	4
4	80X80 PROP		1
5	ISO 10642 - M5 x 10 -- 10N		8
6	ISO 10642 - M6 x 8 -- 8N		2
7	BL-2-1018	Top bolt for benløfter	1
8	BL-2-1020	Glide skinne yderrør POM	4
9	ISO 7380 - M5 x 10 -- 10N		8
10	ISO 7380 - M8 x 10 -- 10N		1
11	la28-750		1
12	BE-7001	Støtteklods for LA28	1
13	BAT-12		1
14	ISO 4762 M10 x 50 --- 32N		1
15	ISO 7045 - M5 x 12 - Z --- 12N		4
16	5325 PJP 075 L51-10		4
17	DIN 1587 - M10 -- NNU		1

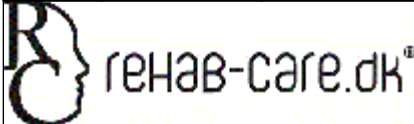

Rev	Date	Note

 <small>Avnvej 10 - 7400-Herring Denmark WWW.rehab-care.dk</small>	Scale: 1:10
	Paper: A3
	Sheet 1 of 1

Description: Dropstativ med motor 750mm	
Surface treatment:	Color: RAL9002
Drawn By: MG	Drawn Date: 04-12-19
Weight: 25945.05	State: - ok
Material:	

 <small>Tolerances ISO 2768-cl</small>	Drawing No.: BL-H5-1000	Rev.
--	-----------------------------------	------