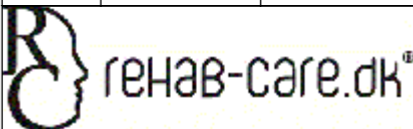



ITEM NO.	PART NUMBER	Description	QTY.
1	BL-H5-100	Dropstativ underdel	1
2	BE-4003	Søjle for benløfter	1
3	BL-2-1008	Glideskinne POM	4
4	80X80 PROP		1
5	ISO 10642 - M5 x 10 - -- 10N		8
6	ISO 10642 - M6 x 8 -- - 8N		2
7	BL-2-1020	Glide skinne yderrør POM	4
8	ISO 7380 - M5 x 10 -- - 10N		8
9	la28-750		1
10	BE-7001	Støtteklods for LA28	1
11	BAT-12		1
12	ISO 4762 M10 x 50 --- 32N		1
13	5325 PJP 075 L51-10	Hjul med bremse ø75	4
14	DIN 1587 - M10 -- NNU		2
15	BL-H5-400	Hjulramme til dropstativ med motor	1
16	DIN 912 M10 x 16 --- 16N		4
17	DR-1043	Drop ophæng	4
18	Hexagon Nut ISO 4032 - M8 - W - N		4
19	DIN 912 M10 x 90 --- 32N		1

Rev	Date	Note
A		

		Scale: 1:10
Avnvej 10 -7400-Herring Denmark WWW.rehab-care.dk		Paper: A3
Sheet 1 of 1		
Description: Dropstativ med motor 750mm		
Surface treatment:		Color: RAL9002
Drawn By: MG	Drawn Date: 04-12-19	
Weight: 25252.24	State: - ok	Material:
 Tolerances ISO 2768-cl		Drawing No.: BL-H5-1000 Rev. A