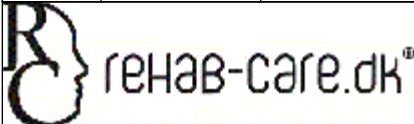



ITEM NO.	PART NUMBER	Description	QTY.
1	L53-H	Glideklods	4
2	BL1-400-V	Søjle motor MOT-61	1
3	BL1-400	Søjle motor MOT-61	1
4	SØ3-1	Aksel for klods	4
5	DIN 912 M10 x 40 --- 40N		1
6	DIN 912 M10 x 20 --- 20N		1
7	DIN 1587 - M10 --NUC		1
8	SØ3-10	Topplade røngten leje	2
9	GAS-02		2
10	GAS-01	Gasfjeder 150N	2
11	GFHANDTAG	Håndtag for gasfjeder	2
12	GFUDLOSERH OVED	Udløserhoved for gasfjeder	2
13	BR2-A214-100	Sengehest for bruseleje	2
14	Pedal-højre	Bremsepedal	1
15	Pedal-venstre	Bremsepedal	1
16	L-4000	Bundramme søjle	1
17	5444-46PJP125R36-32	125mm hjul kort sokkel	4
18	60X30 Flad oval	Plastprop grå 60x30 f-oval	4
19	L-3000-50	Hjul holder	4
20	CO61	Styreboks	1
21	BA21	Batteri	1
22	BR2-25-01-1	Overpart bruseleje	1

Rev	Date	Note

 <small>Avnvej 10 -7400-Herring Denmark WWW.rehab-care.dk</small>	Scale: 1:10
	Paper: A3
	Sheet 1 of 1
Description: Bruseleje opbygning søjle	
Surface treatment:	Color: RAL 9002
Drawn By: MG	Drawn Date: 15-01-20
Weight: 67062.67	State: - ok
Material:	
 <small>Tolerances ISO 2768-cl</small>	Drawing No.: BR-SØ-1
Rev.	