


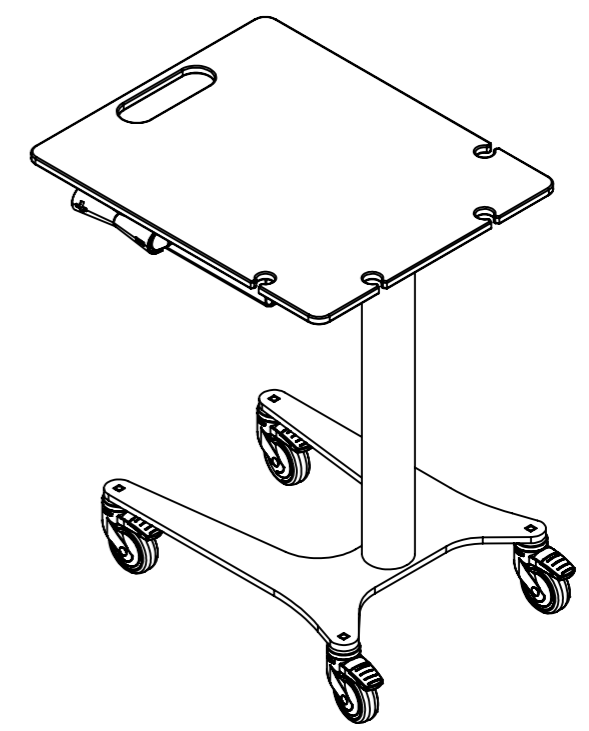

ITEM NO.	PART NUMBER	Description	QTY.
1	DRP-501	Grundplade bord	1
2	DL14	Løfte søjle 3-delt	1
3	DRP-519	Bordplade EPJ	1
4	FS3-smal	Betjening	1
5	BA001	Batteri	1
6	cbd6s	Styring	1
7	DRP-520	Topbeslag for bord	1
8	DRP-533	EPJ bakke	1
9	S70-10x15	Gevind tap for hjul	4
10	5325 PJP 075 L51-10	Hjul med bremse ø75	4



Rev	Date	Note

 rehab-care.dk <small>Avnvej 10 -7400-Herring Denmark WWW.rehab-care.dk</small>	Scale: 1:10
	Paper: A3
	Sheet 1 of 1

Description: EPJ bord med motor 450x550 og papirbakke	
Surface treatment:	Color:
Drawn By: MG	Drawn Date: 19-12-19
Weight: 19320.13	State: - ok
Material:	

 Tolerances ISO 2768-cl	Drawing No.: DRP-680	Rev.