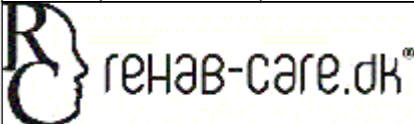



ITEM NO.	PART NUMBER	Description	QTY.
1	PO-1000	Tårn	1
2	PO-2000	Arm	2
3	PO-3001	Løfteplade	2
4	PO-3000	Løftearm	1
5	LA34-400 bund-02	LA34-400	1
6	PO-3005	Aksel for løft	4
7	Jumbo	CBJ styring	1
8	PO-1010	Plast ring bund	1
9	PO-3007	Aksel for arm	4
10	60-65 rørbøsning		2
11	PO-9003	Rør for drejeled	1
12	UAP 150x32-8	Hjul ø150	2
13	PO-7000	Hjulbøjle	1
14	Pin-ø10-62	Pin ø10 L62	1
15	Hexagon socket button head screw ISO 7380 - M8 x 12		14
16	Pin-ø10-32-RF		1
17	Pos-bolt M10x1-rustfri		1
18	PO-9300	Skærm for lift	2
19	RI 1916		8
20	PO-5300	Stol for lift	1
21	PO-9700	Transport hjul for lift	1
22	PO-3008	Aksel for løft	1
23	LA23-motor		1
24	Nut DIN 1587 - M8		2
25	Hexagon socket button head screw ISO 7380 - M5 x 10		2
26	Hexagon socket head cap screw DIN 912 - M6 x 35		1
27	PO-9803	Lukke plade for holderør-top	1
28	PO-9800	Holder for lift til indstøbning	1
29	PO-1500	Løfteplade	1
30	Hexagon socket button head screw ISO 7380 - M10 x 20		1

Rev	Date	Note

 Cypresvej 3 -7400-Herning Denmark WWW.rehab-care.dk	Scale: 1:20
	Paper: A3
Description: Pool lift	
Sheet 1 of 1	
Surface treatment:	Color:
Drawn By: MG	Drawn Date: 15-11-12
Weight: 55011.39	State: - ok
Material:	
 Tolerances ISO 2768-mK	Drawing No.: PO-10000
	Rev.