


Rev	Date	Note
D	26-11-12	Tværrør -2mm
C	19-01-12	200 kg. model


rehab-care.dk
 Cypresvej 3 -7400-Herring Denmark WWW.rehab-care.dk
 Description: **Active lift Cathy**

Scale:	1:10
Paper:	A3
Sheet 1 of 2	

Surface treatment:		Color:	
Drawn By: MG	Drawn Date: 22-03-13		
Weight: 44543.03	State: - ok	Material:	
 Drawing No.: PR-3500-200-S			Rev. D

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	PR-2000	Tower mount	1
2	PR-1500-S	Leg righth	1
3	PR-3100	Trækstang for ben-V	1
4	PR-1000-S	Leg left	1
5	PAF 2825115 P10	Kræbebøsning ø28/25-11mm	6
6	PR-3004	Aksel for ben	2
7	PR-3005	Distance ring for legs	2
8	PAF 15090 P10	Kræbebøsning ø17/15-9mm	2
9	PR-4000-S	Tower for stand up lift	1
10	60X30 PROP		2
11	PR-5004	Aksel til løftearm	1
12	LA34-200-02 bund	Linak act. LA34-200mm	1
13	PAF 14090 P10	Kræbebøsning ø14/12-9mm	3
14	AF-12-10,5	Afstandsring ø12x10,5	3
15	Pin-ø10-47	Pin-ø10-47	1
16	Jumbo	CBJ styring	1
17	PR-1006	Leg adjustm nut-12mm	2
18	PR-1010	Leg adjustm nut-3mm	2
19	DIN 912 M10 x 35 --- 35N		2
20	Hexagon socket head cap screw DIN 912 - M8 x 12		2
21	PR-1008	Afstandsør	2
22	LA23-bund-50mm	LA23-50 akt	1
23	PR-3007	Gear plate for legs	1
24	Hexagon socket button head screw ISO 7380 - M10 x 20		5
25	Hexagon socket button head screw ISO 7380 - M10 x 40		4
26	PU-B-295-S	Kneepad	2
27	PR-3000	Trækstang for ben-H	1
28	Hexagon Thin Nut ISO 4035 - M10 - N		2
29	Circlip DIN 471 - 10 x 1		2
30	PR-4500-S	Holder for knee supp.	1
31	PR-6000-S	Foot plate frame	1
32	PR-6008-S	Foot plate alu.	1
33	ISO 8678-M12x80-30-N		6
34	Prevailing torque type hexagon nut ISO 7040 - M12		6
35	5940UAP100P30-11	Tente wheel ø100	2
36	ZS-12x1-75	Pos bolt for adjust	1
37	ISO 4029 - M6 x 8-N		1
38	Sticker	CE-mark and serie no	1
39	Hexagon socket head cap screw DIN 912 - M5 x 12		3
40	DIN EN ISO 7040 - M10 - N		2
41	PR-5600-S	Lifting arm	1
42	Pin-ø10-42-star	Pin for atc top	1
43	STAR-LOCK		2
44	DIN 912 M16 x 25 --- 25N		2
45	5947UAP100P30-11-kick-p	Wheel with brake	2
46	Nut DIN 1587 - M5		2
47	PR-9100-S		1
48	PR-9101-S	Skærm for ben højre	1
49	PR-9102-S	Skærm for ben venstre	1

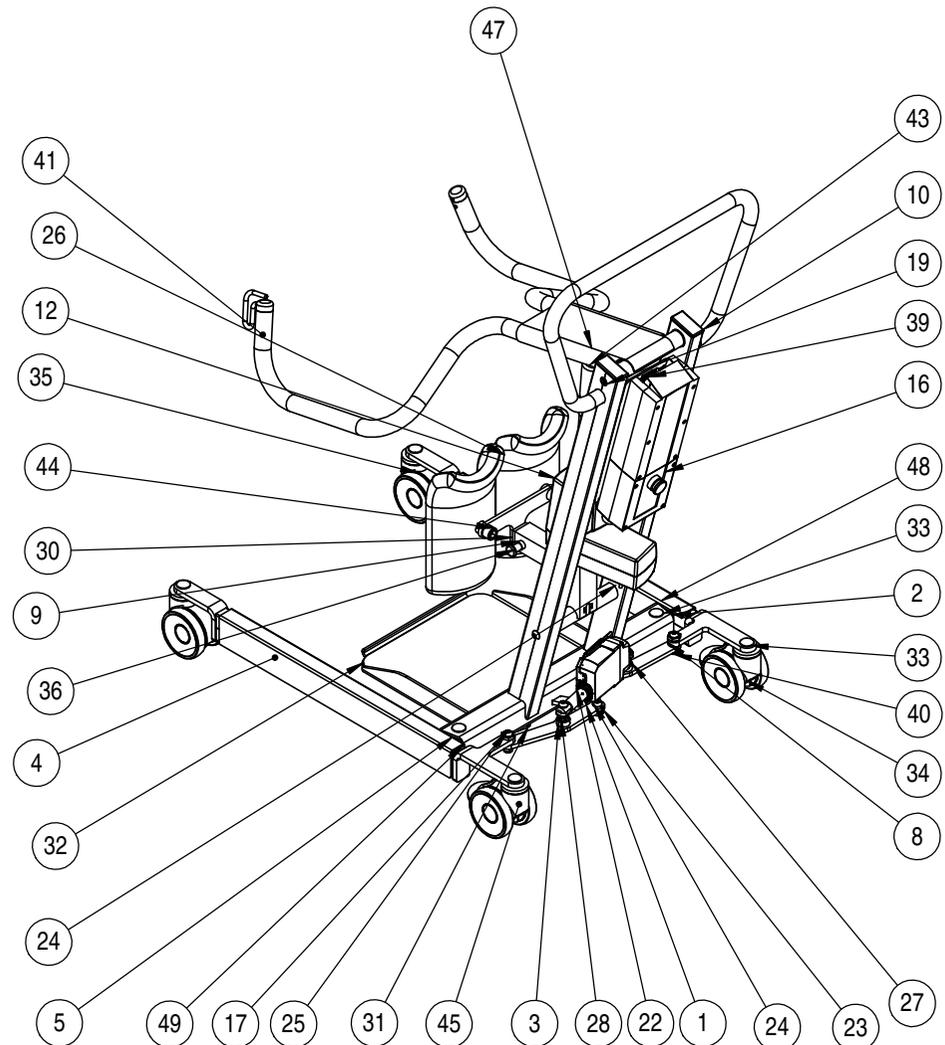

Rev	Date	Note
D	26-11-12	Tværrør -2mm
C	19-01-12	200 kg. model

 rehab-care.dk <small>Cyprusvej 3 -7400-Herring Denmark WWW.rehab-care.dk</small>	Scale: 1:10
	Paper: A3
	Sheet 2 of 2

Description: **Active lift Cathy**

Surface treatment:	Color:
Drawn By: MG	Drawn Date: 22-03-13
Weight: 44543.03	State: - ok
Material:	

 Tolerances ISO 2768-mK	Drawing No.: PR-3500-200-S	Rev. D
---	--------------------------------------	------------------