


Bred hjulbom til sengebredde 120 -140 cm

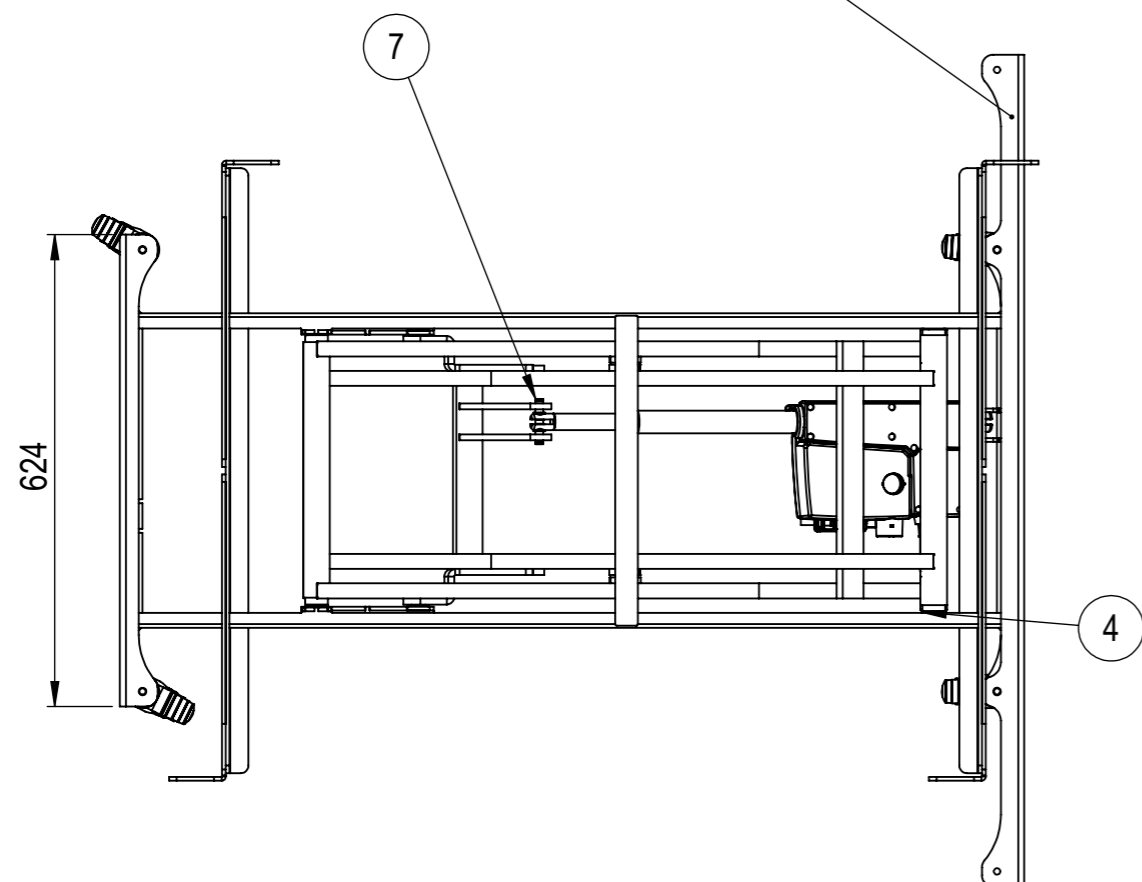

ITEM NO.	PART NUMBER	Description	QTY.
1	L5-B	Løfte travers	1
2	L4A	Saks udv.	1
3	L6-A	Løfte laske	2
4	L53-H	Glideklods	4
5	M5	Saks indv.	1
6	SE-1000	Bundramme sengeløft	1
7	Pin-ø10-54	Pin ø10 x 54	2
8	SE-4100	Topramme sengeløfter	1
9	S70-10x15	Gevind tap for hjul	4
10	PAF 201812 P10	Krave bøsning	4
11	DIN 912 M12 x 20 --- 20N		2
12	LA31 bund 250-løft	Aktuator for løft 250mm	1
13	LA31 top 250		1
14	CB09	Linak styreboks	1
15	SE-1015	Hjul bom bred	2
16	SE-1016	Sengebeslag	4
17	5325 PJP 075 L51-10	Hjul med bremse ø75	4

Rev	Date	Note


 rehab-care.dk Cyprusvej 3 -7400-Herning Denmark WWW.rehab-care.dk	Scale: 1:10
	Paper: A3
	Sheet 1 of 2

Description: **Sengeløfter**

Surface treatment: _____ Color: _____

Drawn By: MG Drawn Date: 21-05-14

Weight: 45811.57 State: - ok Material: _____

 Tolerances ISO 2768-mK	Drawing No.: SE-11000	Rev.