


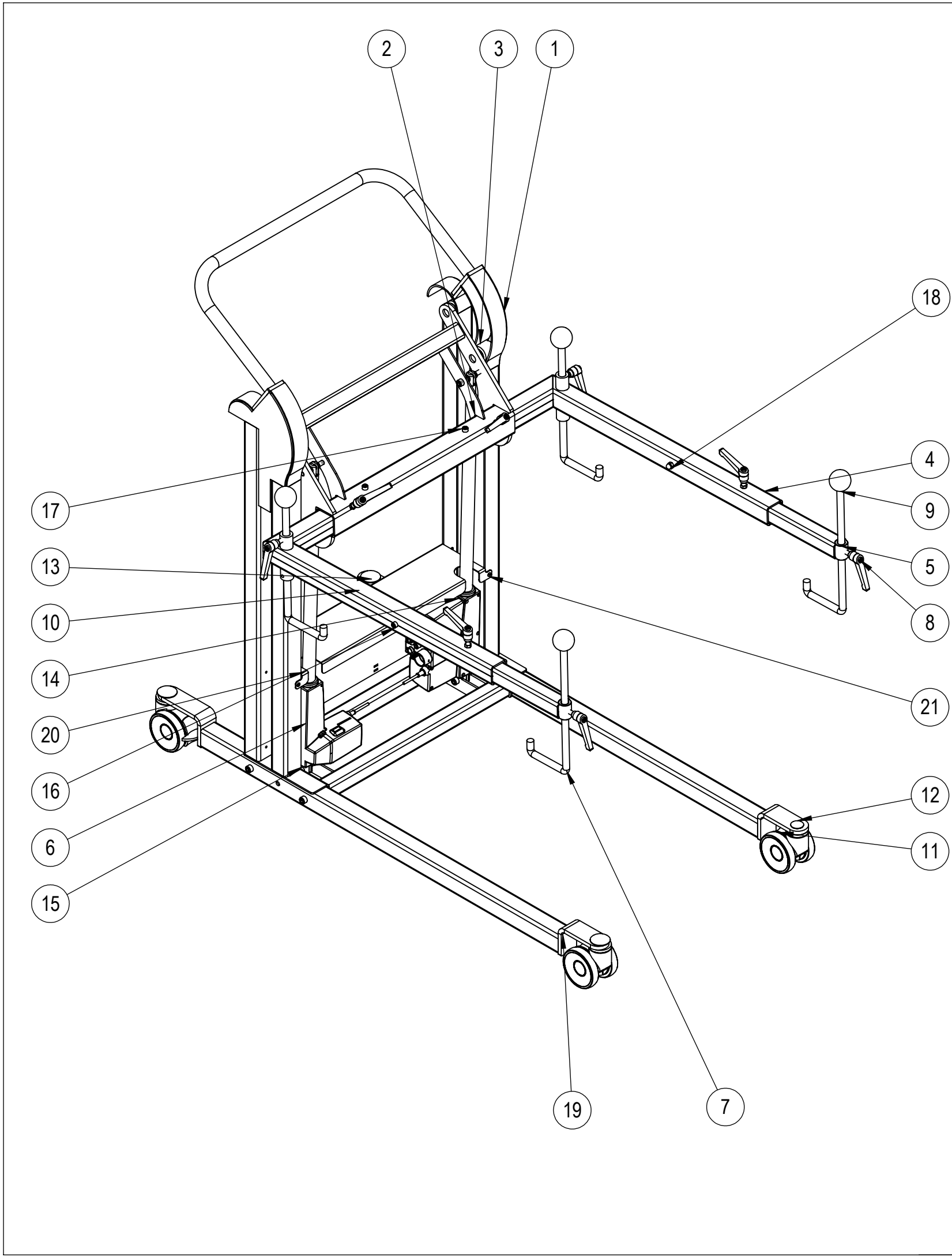

ITEM NO.	PART NUMBER	Description	Default/ QTY.
1	SV14	Tårn til sengevender	1
2	SV2	Bom for sengevender	1
3	SV10	Hjul til sengevender	4
4	SV4-400	Arm for sengevender	2
5	SV5	Udtræks arm	2
6	LA31 bund 500 s		2
7	SV9	Krog for sengevender	4
8	Ganter greb M8		8
9	Kugle ø50-M10 sort	Kugle ø50/M10	4
10	5947UAP100P30-11	Tente wheel ø100 brake	2
11	5940UAP100P30-11	Tente wheel ø100	2
12	ISO 8677-M12x80-36-N		1
13	square neck bolt 01_iso		3
14	CB09	Styreboks	1
15	DIN 912 M10 x 70 --- 32N		8
16	DIN 912 M8 x 60 --- 28N		2
17	DIN 912 M8 x 70 --- 28N		2
18	DIN 1587 - M8 --NNU		4
19	SV15	Hjul ophæng	2
20	SV1-5	Dækplade	1
21	Popnitte ø4,8x16		6
22	SV16	Håndtag for sengevender	1

Rev	Date	Note
A	28-01-13	Nyt setup og hjul
		Scale: 1:20
Cypresvej 3 -7400-Herning Denmark WWW.rehab-care.dk		Paper: A3
Description: Ergon Sengevender		Sheet 1 of 2
Surface treatment:		Color:
Drawn By: MG	Drawn Date: 17-12-15	
Weight: 81736.55	State: - ok	Material:
	Drawing No.: SV 400	Rev. A