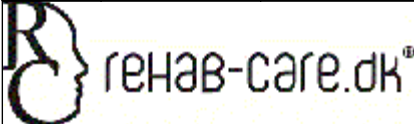



ITEM NO.	PART NUMBER	Description	QTY.
1	U6	Rygstykke	1
2	F1-100	Ben del	1
3	LA31 bund 200-ryg-2	Linak motor MOT-08	2
4	LA31 top 200		2
5	L53-H	Glideklods	4
6	BL1-400-V	Søjle motor MOT-61	1
7	BL1-400	Søjle motor MOT-61	1
8	CB06T	Styreboks	1
9	2474DYK125R26-28S45	Hjul 125mm	4
10	L1-B-SØ	Hjulramme søjle	1
11	W6-1000	Topramme	1
12	SØ3	Top plade til søjle	2
13	SØ3-1	Aksel for klods	4
14	SØ29	Holder for styring	1
15	DIN 912 M10 x 40 --- 40N		1
16	DIN 1587 - M10 -- NUC		1
17	BA18	Batteri	1
18	CG8	Ryg del hynder	1
19	U-11	Ben-del hynde	1

Rev	Date	Note
		Scale: 1:8 Paper: A3 Sheet 1 of 1
Description: 2-delt ULM søjle		
Surface treatment:		Color:
Drawn By: MG	Drawn Date: 17-12-15	
Weight: 135424.52	State: - ok	Material:
 Tolerances ISO 2768-mK		Drawing No.: SØ-ULM Rev.