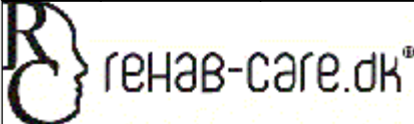



ITEM NO.	PART NUMBER	Description	QTY.
1	L53-H	Glideklods	4
2	BL1-400-V	Søjle motor MOT-61	1
3	BL1-400		1
4	CB06T	Styreboks	1
5	2474DYK125R26-28S45	Hjul 125mm	4
6	L1-B-SØ	Hjulramme søjle	1
7	W6-1000	Topramme	1
8	SØ3	Top plade til søjle	2
9	SØ3-1	Aksel for klods	4
10	SØ29	Holder for styring	1
11	DIN 912 M10 x 40 --- 40N		1
12	BA18	Batteri	1
13	W6-200	Liggeflade beslag	1
14	CG4-100	Gyn sæde	1
15	U6	Rygstykke	1
16	KOL2KB3-180-495-001-300N		3
17	F11	Låsehus nakke fysleje	4
18	Farsfad 30x20x6		1
19	L36	Forlænger til gasfjerder	2
20	F10	Udløserhåndtag fys	2
21	F3	Ryg stykke	1
22	F2	Hoved-stykke	1
23	GAS-17		1
24	GAS-02		1
25	F12	Holder til gas	1
26	U8	Udløserbøjle 2 gas	1

Rev	Date	Note

	Scale: 1:10
	Paper: A3
Sheet 1 of 1	
Description: 4-delt fys-leje søjle	
Surface treatment:	Color:
Drawn By: MG	Drawn Date: 17-12-15
Weight: 77794.08	State: - ok
Material:	
	Drawing No.: SØ600
Tolerances ISO 2768-mK	Rev.