





Styring med batteri backup.
Farve: RAL 9002.

 rehab-care.dk Avnvej 10 -7400-Herring Denmark WWW.rehab-care.dk	Scale:	1:10
	Paper:	A3
	Sheet 1 of 4	
Description:		
Surface treatment:		Color:
Drawn By: MG	Drawn Date: 28-10-17	
Weight: 131398.55	State: - ok	Material:
 Tolerances ISO 2768-cl	Drawing No.: SØ100-02-02	Rev.






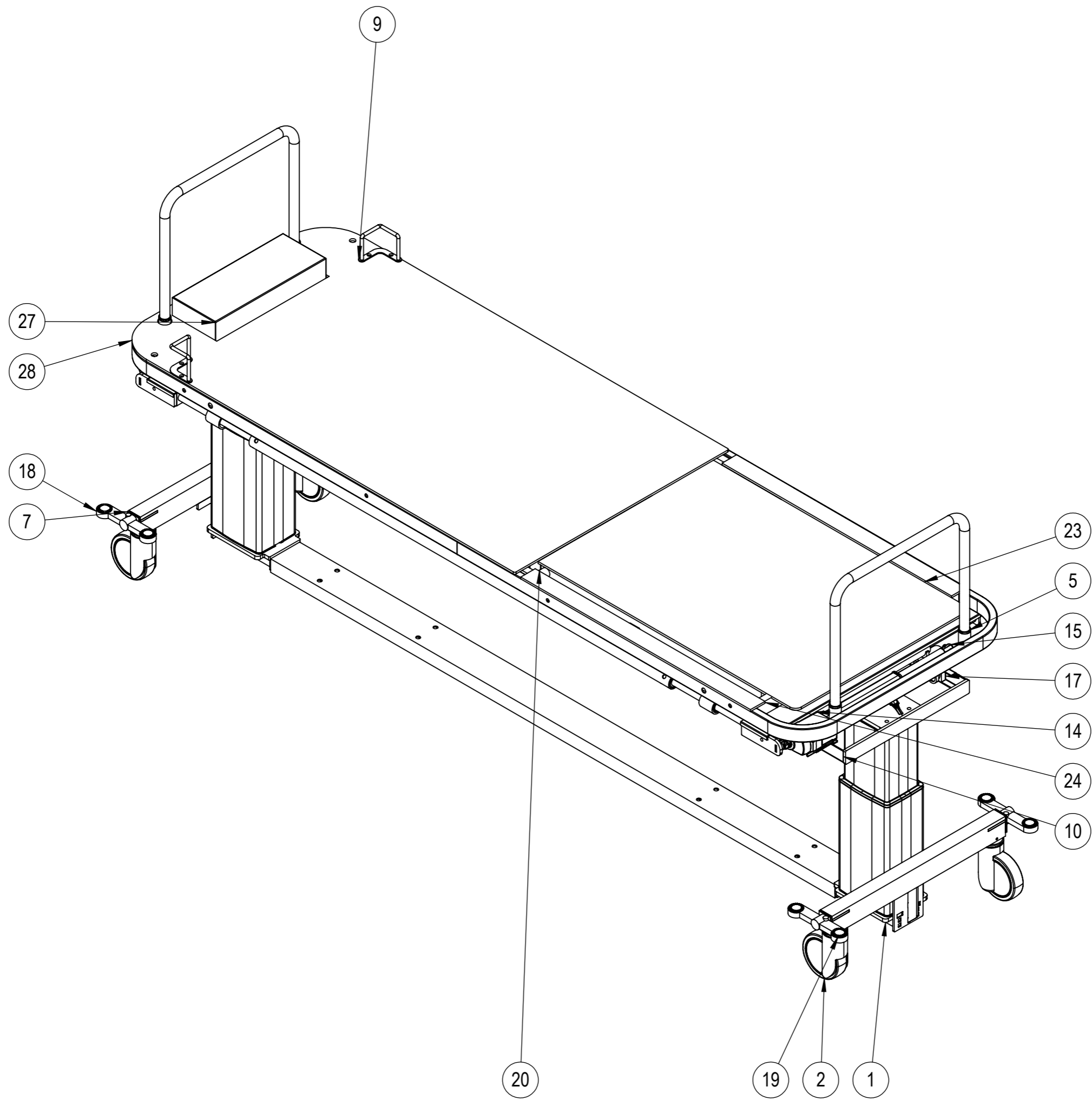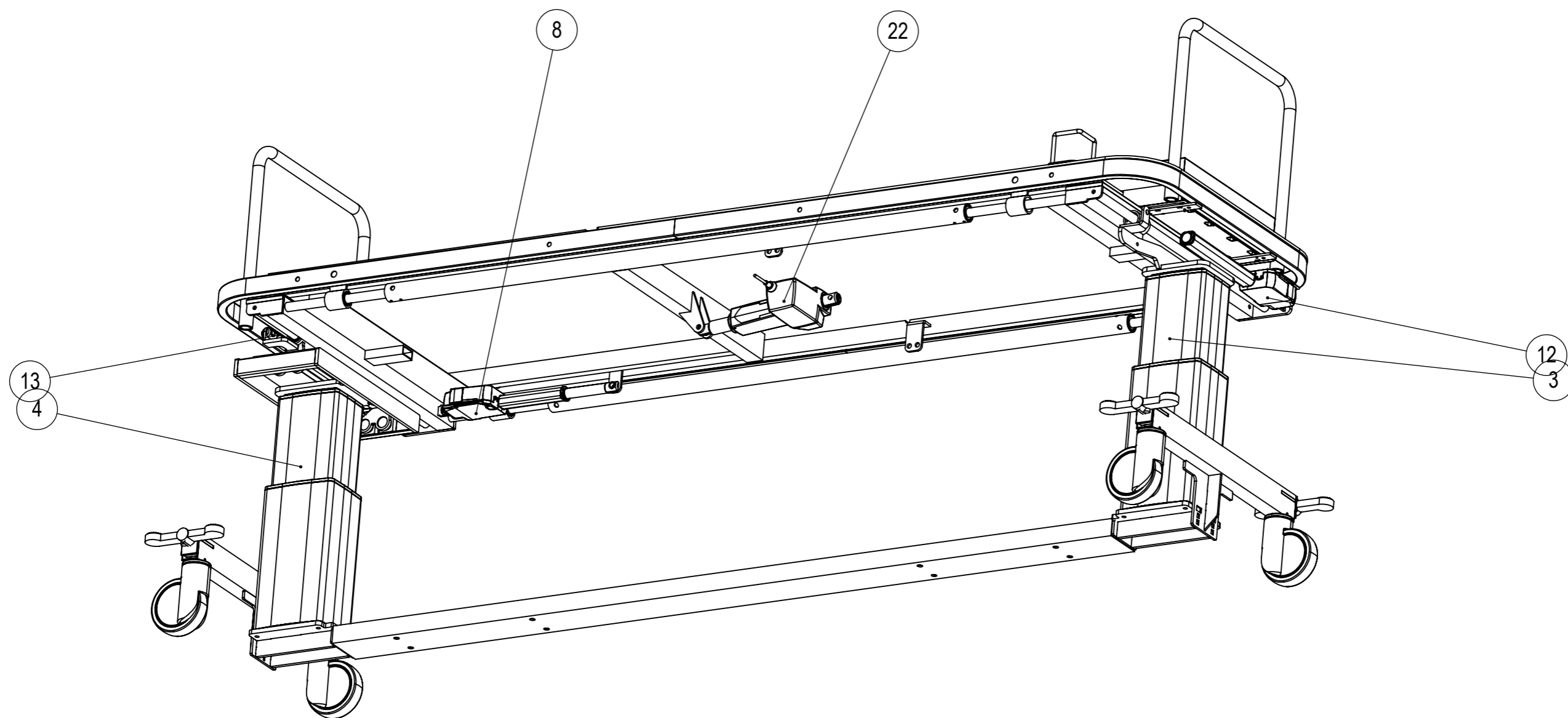

12: Søjle 400 mm slaglængde.
45: LA23-300 1500N indbygn. 418 mm
60: LA23-200 1500N indbygn. 318 mm
74: LA31-150 6000N indbygn. 323 mm

1 stk. CB20 m/batteri
1 stk. Netkabel rød.
1 stk. HB75
Kabler: 2 stk. 2500mm, 1 stk. 2000mm

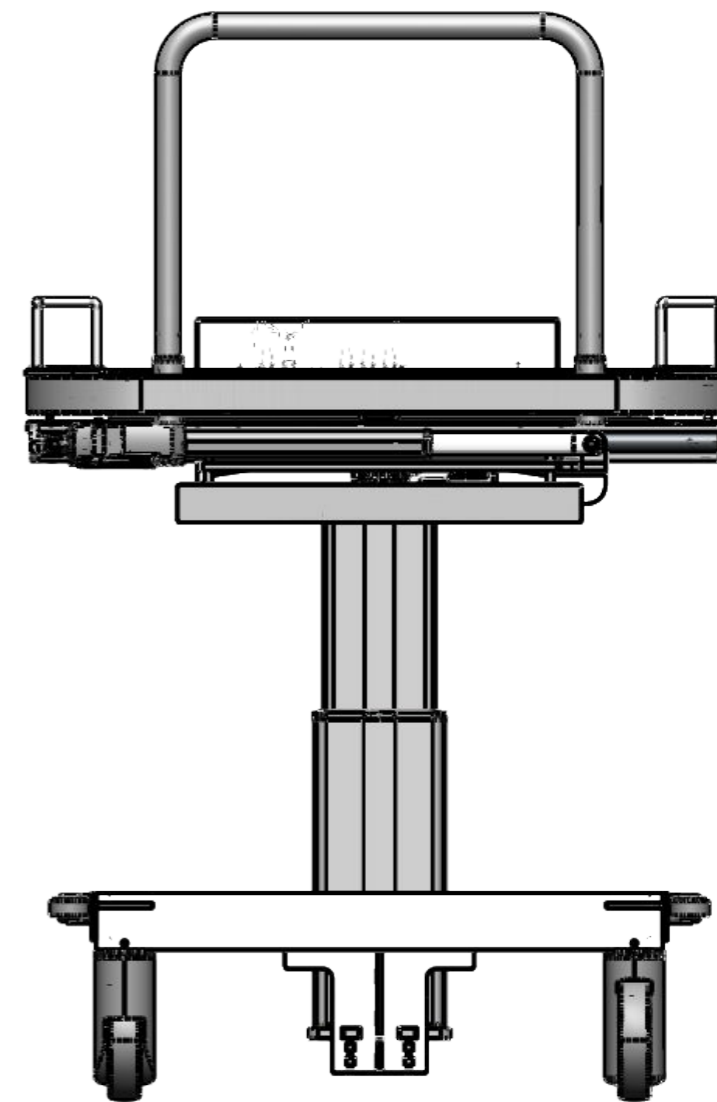ITEM NO.	PART NUMBER	DESCRIPTION	Default/QT Y.
1	SØ11-02	Hjulramme for flydeleje	1
2	2044XSX125R36-32S30-02		4
3	BL1-400-V-02	Linak BL1	1
4	BL1-400-02	Linak BL1	1
5	SØ13	Gavlbojle	2
6	L41-02	Sekskantjern til bremse L=663mm	2
7	50x40 PROP m.hul-02	Rørprop 50x40	4
8	SL-12500-02	Topramme komplet	1
9	U1000-styr	Madrasstyr for montage	2
10	SL-1800-02	Bespænding venstre	1
11	SL-1500-02	Bespænding højre	1
12	LA23-300-1500N		2
13	LA31 top 300	Indgår ikke	1
14	Bearing-SBE20UU-Rollco		2
15	SL-1900-02	Slæde for transport	1
16	SL-1200-02	Holder for kuglebøsning	2
17	L53-H	Sliding block	4
18	Pedal-højre	Bremsepedal	2
19	Pedal-venstre	Bremsepedal	2
20	SØ23-02	Rygstykke	1
21	LA31 top 150	Indgår i LA31	1
22	LA31 bund 150 s		1
23	SØ22-14-02	Laminatplade ryg	1
24	SØ22-15-02	Laminatplade ryg fast	2
25	SKA28	Medbringer	2
26	CB20	Linak styreboks komplet	1
27	SØ26-02	Skærm for styreboks	1
28	SL-1070-02	Laminatplade 6mm	1



 rehab-care.dk <small>Avnvej 10 -7400-Herring Denmark WWW.rehab-care.dk</small>	Scale: 1:10
	Paper: A3
	Sheet 4 of 4
Description:	
Surface treatment: _____ Color: _____	
Drawn By: MG	Drawn Date: 28-10-17
Weight: 131398.55	State: - ok
Material: _____	
 Tolerances ISO 2768-cl	Drawing No.: SØ100-02-02
Rev. _____	