



ITEM NO.	PART NUMBER	Description	QTY.
1	L-1000	Undervogn saks	1
2	U1-A2		1
3	U31		1
4	U32		1
5	LA31 bund 150 v	Aktuator 150mm sp-line	2
6	LA31 top 150		2
7	CB09	Linak styreboks	1
8	L8-A	Trendelenburg travers	1
9	LA31-3 bund	Linak aktuator 100mm MOT-48	1
10	LA31 top		1
11	CB12	Styring	1
12	U-33-polst		1
13	U-34-polst		1
14	U-35-polst		1
15	Pedal-højre	Bremsepedal	1
16	Pedal-venstre	Bremsepedal	1
17	L53-H	Glideklods	2
18	2474DYK125R26-28S45	Hjul 125mm	4

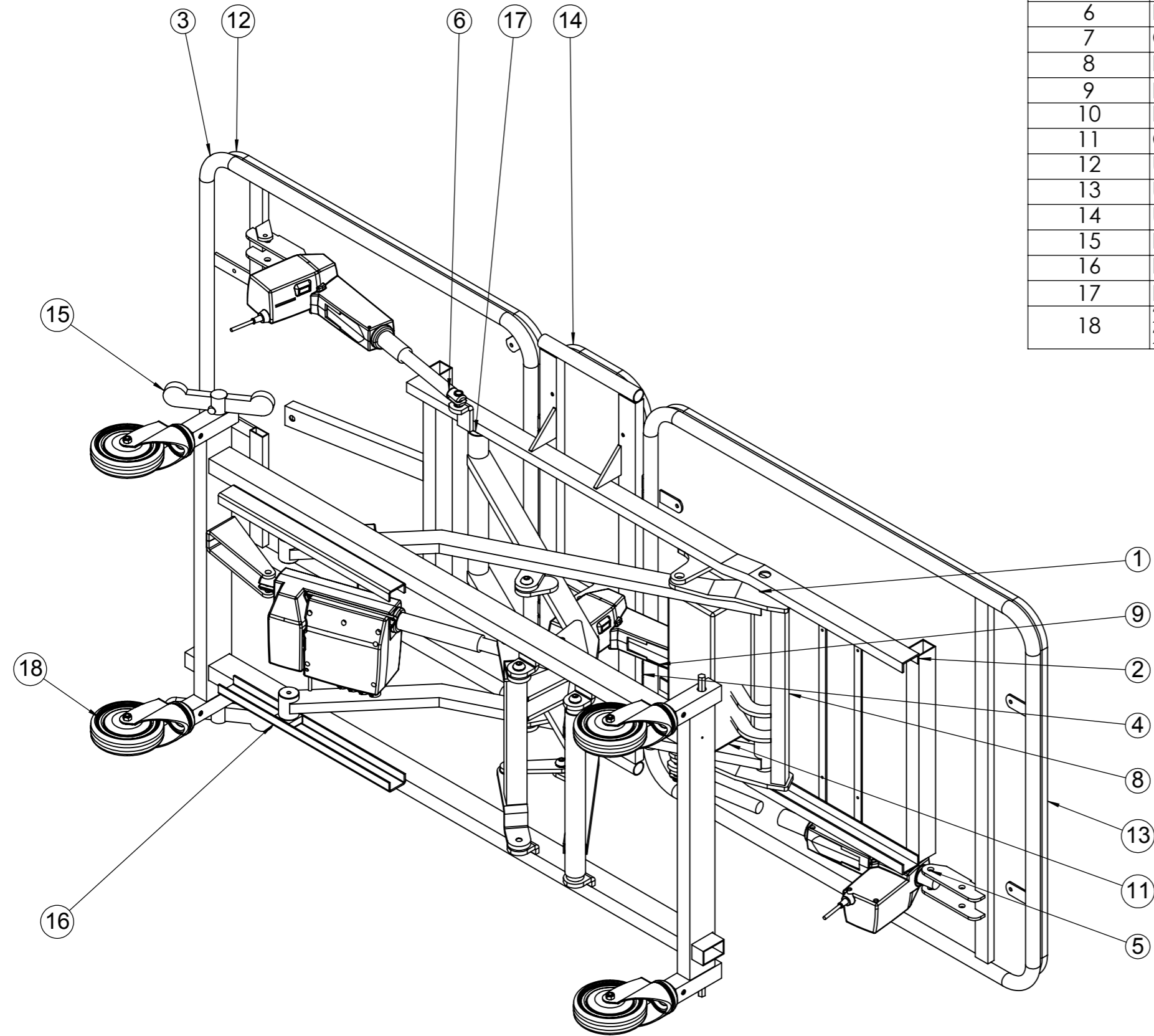


Rev	Date	Note

 rehab-care.dk <small>Cypresvej 3 -7400-Herring Denmark WWW.rehab-care.dk</small>	Scale: 1:8
	Paper: A3
	Sheet 1 of 1

Description: **Ben leje 4 motor med trendeleburg 80cm**

Surface treatment:	Color:	
Drawn By: MG	Drawn Date: 17-12-15	
Weight: 100782.44	State: - ok	Material:

 Tolerances ISO 2768-mK	Drawing No.: U-2000	Rev.
---	----------------------------	------