



ITEM NO.	PART NUMBER	Description	QTY.
1	L53-H	Glideklods	2
2	2044UAX200R26-28S45		4
3	W6-1000	Topramme	1
4	Fenderhjul 100		4
5	SH-200-V	Sengehest komplet venstre	1
6	L16-6	Afstandsrør L10,5	2
7	DIN 912 M10 x 50 --- 32N		5
8	SH-200-H	Sengehest komplet højre	1
9	L26-9	Låseplade for SH	4
10	Pos-bolt M12-ZS	Pos bolt M12	2
11	L26-100	Afstandsrør SH	2
12	L-4000	Bundramme søjle	1
13	SKA28-1	Medbringer bremse	2
14	Pedal-højre	Bremsepedal	1
15	Pedal-venstre	Bremsepedal	1
16	60X30 Flad oval	Plastprop grå 60x30 f-oval	4
17	L42-20	Bremsestang til pedal	1
18	L42-10	Bremsestang L=580	1
19	L-3000-50	Hjul holder	4
20	LA40-180	Rygmotor MOT-80	2
21	L40	Bøsning plast	4
22	CO61	Styreboks	1
23	L73	Topramme akut leje	1
24	L82	Laminatplade til akut	1
25	L26-540	Ryg stykke	1
26	L81	Ryg plade 3-akut	1
27	M10-K-spec-bolt	Bolt M10 med bryst kort	1
28	L20-46	Madrassyk	1
29	L26-560	Vangebeslag	1
30	L20-47	Mardassyk	2
31	L7-14	Dorn for sengehest	4
32	ø32 prop	Prop plast ø32	4
33	L7-13	rør buk gavlbøjle	2
34	BL1-400	Søjle motor MOT-61	1
35	BL1-400-V	Søjle motor MOT-61	1
36	SØ3-1	Aksel for klods	4
37	W6-24	Holder for styring	1
38	SØ3	Top plade til søjle	2
39	Circlip DIN 471 - 32 x 1.5		4
40	5444-46PJP125R36-34	Hjul ø125	4

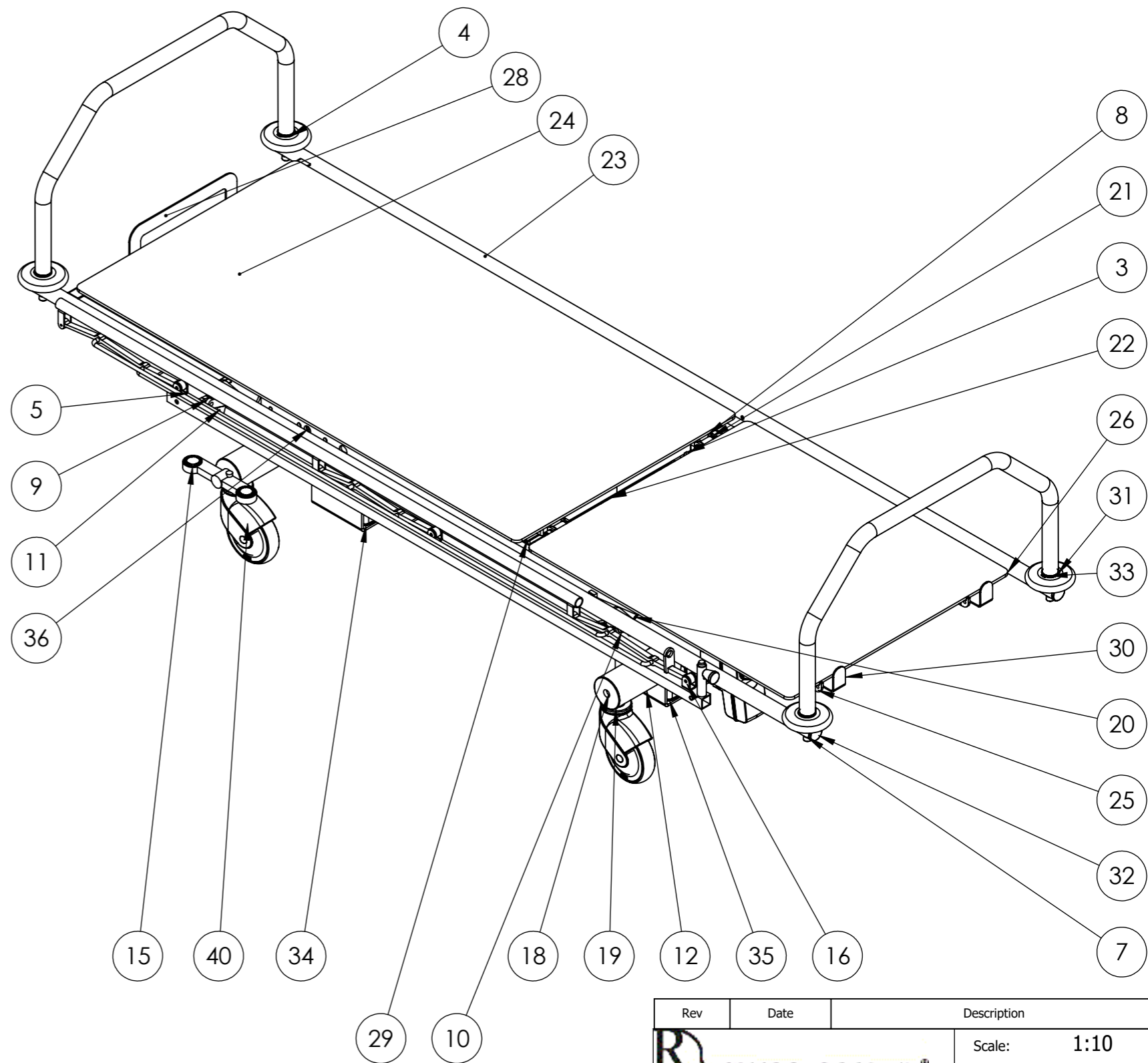

Rev	Date	Description
		Scale: 1:10
		Paper: A3
Sheet 1 of 2		
Description: Hvile transportleje 2-delt søjle		
Surface treatment:		Color:
Drawn By: MG	Drawn Date: 10-01-20	
Weight: 99119.20	State: - ok	Material:
 Tolerances ISO 2768-cl		Drawing No.: <b>Akut-12000</b>
		Rev.

