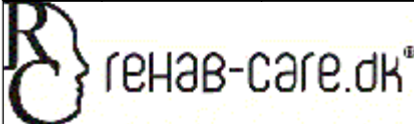



ITEM NO.	PART NUMBER	Description	QTY.
1	5325 PJP 075 L51-10	Hjul med bremse ø75	5
2	DR-1029	Lås for dropstativ	1
3	ø40 håndskruex15		1
4	S70-10x15	Gevind tap for hjul 15mm	5
5	DIN 1587 - M10 --NNU		5
6	PU-B-295-S	Kneepad	1
7	kugle-led	Kugle-led	1
8	DR-1030	Fod til dropstativ	1
9	DR-1039	Rør for drop	1
10	DR-1054	styr	1
11	L-3000-64	Gevind for hjul	1
12	DIN 912 M10 x 50 --- 32N		1
13	Dupsko 25	Dupsko ø25	1
14	Popnitte ø4x13 rustfri		1
15	DR-4100	stang til benholder	1

Rev	Date	Note
B	17-01-19	rustfri bund

 Cypræsvej 3 -7400-Herring Denmark WWW.rehab-care.dk	Scale:	1:4
	Paper:	A3
	Sheet 1 of 1	

Description: <b>Fod holder</b>		
Surface treatment:	Color:	
Drawn By: MG	Drawn Date:	
Weight: 7282.18	State: - ok	Material:
 Tolerances ISO 2768-mK	Drawing No.: <b>DR-15000</b>	Rev. <b>B</b>