05-2017







CE

1. Contents

1. Contents	2
1. Introduction	3
2. Unpacking	3
3. General Safety Regulations	4
4. Area of Use	5
5. Use of the Chair	5
6. Expected lifespan of the product	6
7. Equipment and Accessories	6
3.Brakes	6
9. Manoeuvring instructions	7
10. Handling of Unit Parts	7
11. Cleaning	8
12. Maintenance	8
13. Service	8
14. Transport and storage	8
15. Disposal	9
16. Guarantee	9
17. Technical Specifications	10
18. Symbol Explanations	11
Annex A Periodic inspection	11
Annex B Daily inspection before use	13
Annex C Guidance, Yearly Inspection of chair	14
Annex D Drawing	16
Annex E Spare parts list	17

1. Introduction

05-2017

RehabCare congratulates you on your new equipment.

RehabCare Toilet and Bath Chair is a modern and up-to-date product based on our long experience in the field. This Chair enables you to enhance function, quality and handling in one optimal solution. Through continual development in all phases of our production process we endeavor to ensure conscientiousness and quality. Therefore Rehab-Care can guarantee a very reliable and functional high-quality product. Satisfied customers are the primary goal in our product development.

The Chair fulfils requirements in respect of the medical directive for class 1 products (Medical Devices Regulation (EU) 2017/745 - MDR), EN 60601-1: 2015, EN60601-1-2: 2015, ISO 10535: 2006. Thank you for your confidence in our products and for your collaboration.

Sincerely

RehabCare ApS.

Manufacturer:



Agent:

2. Unpacking

Carefully unpack the chair. Check that all parts and accessories are included. Check the chair for eventual transport damage. We recommend that the chair be cleaned before use, see cleaning instruction.

Contact the agent or manufacturer if the chair is damaged or non-functioning.

05-2017

3. General Safety Regulations

\triangle	Warning! Do not use a high-pressure cleaner on the product.
\triangle	Warning! Do not use chemicals, and carry out a yearly service inspection for damage
\triangle	Warning! Foreign bodies in the wheels can damage/destroy the chair's wheels.
\triangle	Warning! Do not use wheels other than those recommended by the manufacturer.
\triangle	Warning! Children must not use the chair without supervision. It is not a toy!
\triangle	Warning! The chair must not lift more than 200 kg
	Warning! Use only the recommended seat and other parts of the chair.
\triangle	Warning! Before and during a lift, ensure there is no danger of crushing for user or helper.
\triangle	Warning! The chair must not be used if a malfunction is suspected.
\triangle	Warning! Do not lift the user higher than necessary, and do not tilt the chair more than necessary.
\triangle	Warning! The chair can overturn if used incorrectly. Personal injury can occur if warnings and instructions are disregarded. Read instructions carefully before using the chair.
	Attention! Deference

Attention! Before use: To avoid injury during handling and use of the chair, the instructions for safety and use should be read and understood.

05-2017

4. Area of Use

The Chair is for use in a private home, care homes and hospitals

In toilet-, bath- and sanitary areas. In the care of wounds, skin and feet. Washing of hair. Dressing and undressing. As transport to toilet/bathroom, etc.

The chair is intended for indoor use. The chairs Safe Working Load (SWL) is made up as follows: Equipment: 25 kg. – User: 175 kg. SWL: 200 kg.



Warning!

The chair must not lift more than 200 kg.

5. Use of the Chair

- The chair is for use for general toilet- and bath situations, transport to and from the toilet, transport between bed and wheelchair, dressing and undressing.
- The helper should be aware of all necessary general and safety instructions before the user is lifted with the chair.
- The chair must only be used in accordance with Instructions for Use and by personnel instructed in its correct use.
- The helper should try the chair together with a colleague before lifting the user.
- The user must be instructed in the use of the chair by the helper before using it for the first time.
- Persons with physical or psychological limitations must not use the product without supervision, or unless they have first been instructed in the proper use of the chair and its equipment.
- Children must be under supervision to ensure that they do not play with the product.
- The helper is responsible for ensuring the chair is not defective; therefore a visual control of the chair should be made before a lift. The Agent should be contacted immediately in case of defects.
- Ensure all wheels are unlocked before moving the chair.

6. Expected lifespan of the product

The product has an expected lifespan of 15 years.

Lifespan is based on the correct use, cleaning, service and maintenance.

If the chair is to be used longer than the above mentioned expected lifespan, it should be assessed by qualified service personnel before future use.

It is the helpers' responsibility to assess whether the product can be used over the expected lifespan.

7. Equipment and Accessories


The chair comprises:

- Frame with 4 wheels
- Manuel adjustment of the seat angle
- Manuel adjustment of the chair height

Optional accessories:

- Footrest with adjustable plate
- Seat cushion with diverse openings
- Legstrap in artificial leather
- Back support in artificial leather Leg rest
- Body/side support
- Safety guard
- Hemi-armrest
- -
- And more. Please see RehabCare's complete accessory list.
- -

8.Brakes

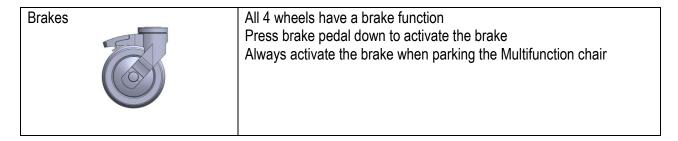

05-2017

9. Manoeuvring instructions

- 1. The chair should be set to the lowest possible position when moving a user, according to the needs of the user.
- 2. By using the pump system, the helper can be close to the user and create a feeling of security
- 3. The chair should be pulled during a move, this gives a smoother ride for the user, and at the same time puts less strain on the helper
- 4. The helper should stand at the side of the chair when turning. The chair should be turned on its own axis; the helper pulling with one hand on the chair's back-/maneuvering guard. When using the chair, it is recommended that the helper uses the whole body and not twists torso and shoulders.
- 5. The helper must always check on the chair's stability and that there is no risk of overturning. Check for obstacles in the chair's path.
- 6. Be prepared for the task: move a wheelchair to the chair, or the chair to a bed.
- 7. Push or pull the chair smoothly; hasty movements are uncomfortable for the user and an abrupt stop can be a safety danger.
- 8. Pull the chair if it must go over a doorstep or other obstacle, this makes the movement smoother and safer.
- 9. Do not lift the chair, this is dangerous for both helper and user.
- 10. Always use the wheel brakes when the user is in the chair.
- 11. Do not raise/tilt the user more than necessary.
- 12. Never stand on the footrests; this could injure the user and/or helper and damage the product.

10. Handling of Unit Parts

Footrests and leg supports. None of the accessories weighs more than 11 kg.

05-2017

11. Cleaning

Metal parts of the chair can be cleaned with cleaning or disinfectant fluid. The chair can be cleaned in accordance to SSI and MRSA. Also with ethanol 70-85%, chlorine 1000-1200 ppm and chlorine cloths (wet serviettes).

The wheels must only be cleaned with products not containing corrosive or damaging materials. The chair is made to be washed in a wash-machine/tunnel.

12. Maintenance

- Wheels must be free of hair, soap, and dirt.
- Yearly service is recommended.
- Moving parts should be checked, tightened and greased.
- Worn chairs should be replaced.

13. Service

The chair should undergo a yearly service, carried out by qualified personnel. The yearly service is compulsory according to standard EN ISO 10535. It is the owner's responsibility to ensure the yearly service is undertaken. A service protocol should be established for the serviced chair, and be signed by the customer after every service.

If the service technician discovers a serious fault during service, it is the technician's duty to remove the chair if the fault constitutes a safety risk.

Service, repairs and other additional work on the chair between services should be noted in the service protocol.

18.1 Service - WHEELS

Wheels should be regularly serviced as necessary, usually tightening. The wheels have closed bearings; therefore greasing is not necessary.

Cleaning fluids containing corrosive or damaging materials must not be used Wheels and parts should be replaced as soon as they no longer run smoothly.


14. Transport and storage

If the chair is not to be used for a long time, it is recommended that it be covered to protect it from dust and dirt.

The chair should be stored in a frost-free environment with normal air humidity (80%).

Requirements for storage and transport:

- Air humidity between 10 and 80%
- Temperature between -30 and 40 degrees C
- Pressure between 700-1100 hPa

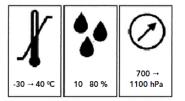

15. Disposal

For disposal, separate the chair into metal waste.

The pump contains oil and should be disposed of according to regulations in force, and must not be disposed of with general waste.

16. Guarantee

There is a 2-year guarantee on the chair.

The guarantee becomes invalid in the following cases:

- 1. Overloading the chair.
- 2. Violent use of the chair.
- 3. Handling fault, wrong use of the chair
- 4. Absence of, or wrong maintenance.
- 5. Damage to the surface, scratches and mechanical interference.

At RehabCare we are continuously endeavoring to improve our products.

We therefore reserve the right to change our products without prior notification.

All measurements given in the drawings or other material are only guidelines, and subject to errors.

05-2017

17. Technical Specifications

17.1 Measurements and Weight

Max. load Lift space, measured from seat or Tilt angle	200 kg 52 – 90 cm. -5 to+30 degrees
Number of lifts with 100 kg.	60 lifts
Chair – outside width	59 cm.
Free height under seat in chair's lowest position	42 cm.
Free height of frame	11 cm.
Length of frame	67 cm.
Wheel diameter	Ø125 mm
Turning radius	Approx43 cm.
Total weight	35 kg
Noise level	Under 50 dB
Surface	zinkfosfateand powder coating
RAL color standard	RAL9006.

18. Symbol Explanations

The following symbols are used on the labels and/or in these Instructions



Warning, see relevant documents



User information



Class 2 el-equipment, double guard steering



Type B equipment (EN 60601-1)



For indoor use



CE approved directive for medical equipment 93/42/EC



Annex A Periodic inspection

A.1 Periodic inspection of the chair should be undertaken as recommended by the manufacturer, at least once a year. Periodic inspection includes visual inspection, especially of the chair's support structure, hoist, brakes, steering, safety and personal aids, as well as a functionality test and any necessary maintenance, e.g. brake adjustment, tightening of screws, etc.

Every inspection must include a test of one (1) lift cycle with the maximum load.

05-2017

A.2 Periodic inspection must be undertaken by a suitably qualified person with knowledge of both the construction, use and maintenance of the chair.

A.3 All observations and remarks should be noted in a logbook held by the person(s) responsible for servicing/maintenance of the chair.

The date when the chair is repaired according to any observations made under the inspection, should also be noted in the logbook.

A.4 The date when the inspection of the chair is undertaken, and the result of the inspection should be noted in the logbook together with the signature of the inspector.

A.5 If the periodic inspection reveals faults, wear or other damage which constitutes a safety risk, the owner should be informed immediately. In the case of a direct safety risk the chair must be taken out of service immediately. The chair must not be used before it is repaired.

A.6 Faults and serious damage which could constitute a safety risk, and which occur between inspections, but which have been corrected and repaired should also be noted in the logbook.

A.7 Faults and damage should be reported to the manufacturer/dealer. This feedback should be noted in the logbook.

Annex B Daily inspection before use

05-2017

Nr.	Check	Remarks
1	All parts in place, no loose parts	
2	The chair has no rust or surface damage.	
3	 The chair shows no sign of other damage. 	
4	Wheels and wheel brake function	
5	Quick guide can be found on the chair	

Daily cleaning

Remove hair, soap and dirt around the wheels Clean the chair with a damp cloth

The chair should be cleaned with cleaning or disinfectant fluid with a brush or sponge. Rinse, then dry thoroughly.

Harsh dissolvent must not be used.

100% chlorine solution must not be used.

When cleaning the wheels, cleaning fluids must not contain corrosive or damaging material

Annex C Guidance, Yearly Inspection of chair

(To be completed by the person responsible, with further control as deemed necessary)

Date::	Product ID / serial nr.:
Carried out by::	
Inspection carried out at:	

Nr.	Check Bemærkning	
	Visuel kontrol af produkt	
1	- Damage	
2	- Distortion	
3	- general damage	
4	- rust	
5	- stability – no loose parts	
6	- wheels run smoothly	
7	- loose wheels	
8	- check lift pump for wear	
Nr.	Check	Remarks
	Additional:	
9	Clean and grease as necessary	
10	Attach label with the date of the next inspection	
11		
12		
Nr.	Check	Remarks
13		
14		
15		
16		
Nr.	Check	Remarks

05-2017


Date: _____ Initials/Signature _____

The yearly inspection can be registered electronically in the product's service log with RehabCare Contact RehabCare for further information

Annex D Drawing


05-2017

D1. Drawing - ErGo Tilt 4000

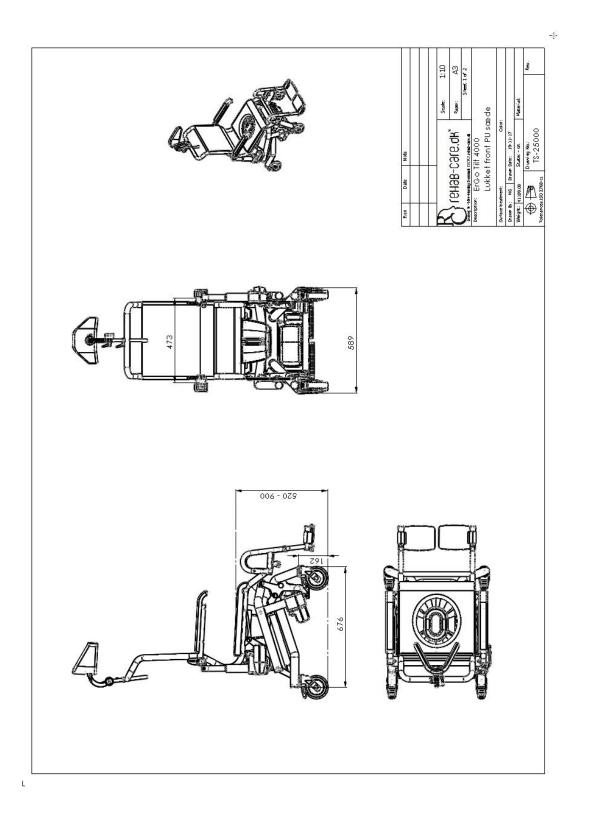

05-2017

Annex E Spare parts list

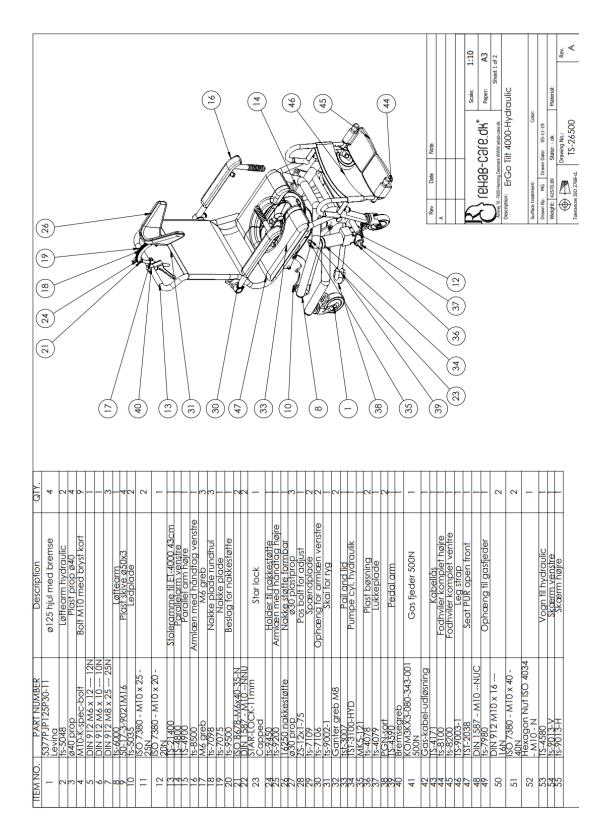

05-2017

Annex F Troubleshooting

Nr.	Fault	Possible cause	Solution
1	Function failure	Check the pump for leaks	contact dealer
2			
3			
4			
5			
6			

Contact the dealer if the chair does not function correctly.